

## PARI BOY® family

Efficient inhalation systems – as individual as the patient's needs

- with PARI LC SPRINT nebuliser – clinically tested in over 50 scientific studies
- efficient lung deposition even with short inhalation times
- with PIF Control – the integrated “training tool” helps patients to learn an effective inhalation technique



**PARI BOY® Junior**  
for babies and small children



**PARI BOY® Classic**  
for use in acute and chronic diseases of the airways of patients aged 4 years and older



**PARI BOY® Pro**  
for use in severe and chronic diseases of the airways of patients aged 4 years and older

## At a glance



### PARI BOY Junior

Item No. 130G1301, Western Europe (de, fr, it, pt, es, en)  
Item No. 130G1306, Eastern Europe (el, hu, pl, sl)



### PARI BOY Classic

Item No. 130G1201, Western Europe (de, fr, it, es, nl, en)  
incl. child and adult mask  
Item No. 130G1206,  
Eastern Europe (bg, el, hr, pl, ro, sl, et, lv, lt)



### PARI BOY Pro

Item No. 130G1001,  
Western Europe (de, fr, it, pt, es, nl, en)  
Item No. 130G1006, Eastern Europe (cs, el, hu, pl, sk, sl)

## PARI Year Packs



### PARI BOY Junior Year Pack

Item No. 023G3311,  
Western Europe (de, fr, it, pt, es, nl, en)  
Item No. 023G3316,  
Eastern Europe (bg, cs, el, hr, hu, pl, ro, sk, sl, et, lt, lv)



### PARI BOY Classic Year Pack

Item No. 023G3211,  
Western Europe (de, fr, it, pt, es, nl, en)  
Item No. 023G3216,  
Eastern Europe (bg, cs, el, hr, hu, pl, ro, sk, sl, et, lt, lv)



### PARI BOY Pro Year Pack

Item No. 023G3011,  
Western Europe (de, fr, it, pt, es, nl, en)  
Item No. 023G3016,  
Eastern Europe (bg, cs, el, hr, hu, pl, ro, sk, sl, et, lt, lv)



PARI GmbH  
Moosstraße 3  
82319 Starnberg, GERMANY  
Tel.: +49 (0)8151 279 220  
Fax: +49 (0)8151 279 6220  
E-Mail: [export@pari.de](mailto:export@pari.de)  
[www.pari.com](http://www.pari.com)

130D0030-A-2019-09



## Nebuliser therapy at home



PARI BOY® family – Tips for use and hygiene

[www.pari.com](http://www.pari.com)

## Usage tips


- 1 Wash your hands thoroughly before each session.



- 2a Attach the mouthpiece or the mask to the nebuliser.



- 2b The baby mask must only be used in conjunction with the baby bend. Adjust the alignment of the bend and the mask according to the position the baby or infant is in.

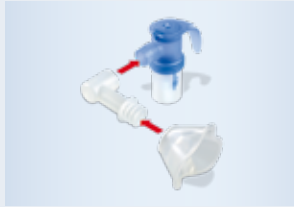

- 3 Connect the nebuliser and compressor with the connection tubing. Connect the compressor to the mains power supply.



- 4 Open the nebuliser cap. Pour in the inhalation solution. Close the nebuliser cap.



- 5a Sit upright in a relaxed position and switch on the device. Take the mouthpiece between your teeth and close your lips around it.



OR

- 5b Make sure that the nebuliser is upright. Gently press the mask against the face so that it fits snugly.



- 6 Breathe in slowly and deeply. When breathing in too fast, PIF Control produces a resistance. If you notice this, stop for a moment, then try to inhale more slowly. When a different noise comes from the nebuliser, this indicates that nebulisation has finished.



- 7 Switch off device. Disconnect the nebuliser from the tubing. Switch the device back on to dry the tubing.



- 8 Disconnect the power plug.



## Hygiene tips

1. Disassemble the nebuliser after each use and **wash** all individual parts in warm water with dishwashing liquid, before rinsing thoroughly in running water.



2. **Disinfect** the nebuliser once a day for 5 minutes in boiling water or for at least 6 minutes in a standard commercial disinfectant, such as one designed for baby bottles.




3. After every cleaning and/or disinfection, place the parts on a dry, clean, absorbent surface and allow to **dry** completely.



4. **Store** all components them in a dry, dust-free place.



5. Air filter, nebuliser and connection tubing should be **replaced** every year. These components are included in the **PARI Year Packs**, which are available from your PARI dealer. Components may have to be replaced sooner if they are damaged or if contamination cannot be removed.

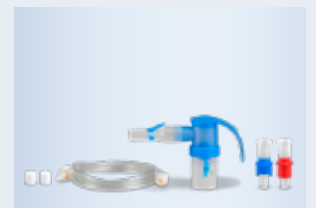

Quick Reference Guide  
For more details, please  
read and follow  
the Instructions for Use.