

PARI COMPACT2

- Robust & reliable compressor device
- Includes the clinically-proven PARI LC PLUS nebuliser, mouthpiece and child mask
- Easy to clean
- Made in Germany – 100 % quality control



You can find more information on www.pari.com.

At a glance




PARI COMPACT2

Inhalation System
with PARI LC PLUS nebuliser, mouthpiece and child mask

(IFU: de, en, es, fr, it, nl)
Item No.: 152G1001

(IFU: cs, el, et, hr, hu, pl, ro, sk, sl)
Item No.: 152G1006

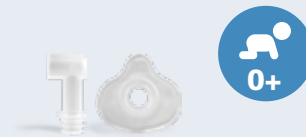

+ For babies and small children:

PARI Baby Set COMPACT2/BASIC

PARI BABY mask size 2 and BABY
bend for children aged 1 – 3 years

Item No.: 041G0706



Annual replacement set:

PARI COMPACT2 Year Pack

PARI LC PLUS nebuliser, mouthpiece,
connection tubing and air filter for
PARI compressor type 052 and 152

(IFU: de, en, es, fr, it, nl)
Item No.: 022G3511

(IFU: cs, el, et, hr, hu, pl, ro, sk, sl)
Item No.: 022G3516



PARI GmbH
Moosstraße 3
82319 Starnberg, GERMANY
Tel.: +49 (0)8151 279 220
Fax: +49 (0)8151 279 6220
E-Mail: export@pari.de
www.pari.com

152D0009-A-2019-09



Proven effective

in the treatment of

lung disease



PARI COMPACT2 – Tips for use and hygiene

www.pari.com

Usage tips

1. Wash your hands thoroughly before each inhalation session.




2. Detach the inspiratory valve from the nebuliser.



3. Pour in the inhalation solution.



4. Attach the inspiratory valve and mouthpiece or mask. When using a mask read the nebuliser instruction for use.

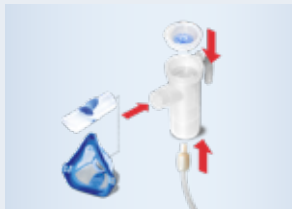

5. Connect the nebuliser and compressor with the connection tubing. Connect the compressor to the mains power supply.



6. Sit upright in a relaxed position and switch on the device.



7. Hold the mouthpiece between your teeth and close your lips around it.




8. Breathe in through the mouthpiece slowly and deeply, then breathe out in a relaxed manner. When a different noise comes from the nebuliser, this indicates that nebulisation has finished.



9. Switch off device. Disconnect the nebuliser from the tubing. Switch the device back on to dry the tubing.



10. Disconnect the power plug.

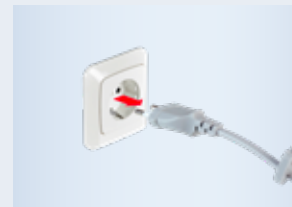

Hygiene tips

1. Disassemble the nebuliser after each use and **wash** all individual parts in warm water with dishwashing liquid, before rinsing thoroughly in running water.



2. **Disinfect** the nebuliser once a day for 5 minutes in boiling water or for at least 6 minutes in a standard commercial disinfectant, such as one designed for baby bottles.




3. After every cleaning and/or disinfection, place the parts on a dry, clean, absorbent surface and allow to **dry** completely.



4. **Store** all components them in a dry, dust-free place.



5. Air filter, nebuliser and connection tubing should be **replaced** every year. These components are included in the **PARI Year Packs**, which are available from your PARI dealer. Components may have to be replaced sooner if they are damaged or if contamination cannot be removed.

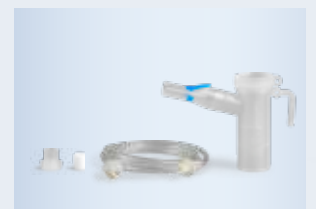

Quick Reference Guide
For more details, please
read and follow
the Instructions for Use.